

# LOCAL MEASUREMENT QUICK REFERENCE

MX3

HYDRATION  
TESTING SYSTEM



# APP MEASUREMENT QUICK REFERENCE



HYDRATION  
TESTING SYSTEM


1

For directions on  
how to turn on device  
and take samples,  
see other side.




Select **APP**

2

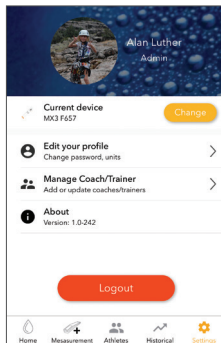


**Setting Tab**  
Connect to  
MX3 Device

3

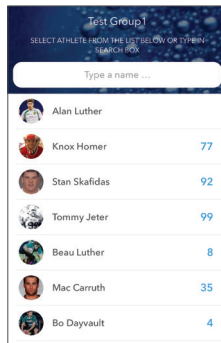


**Measurement Tab**  
Select Athlete

4

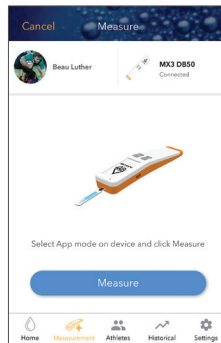


**Measurement Tab**  
Insert MX3 Sensor  
Select Measure  
Apply Sample

5

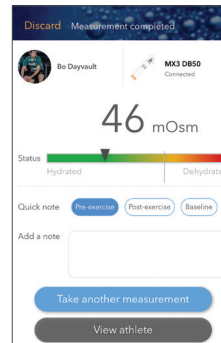

**Measurement Tab**  
Read Result  
Eject Sensor

6

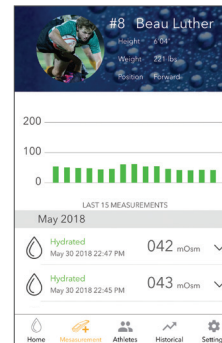

**Athletes Tab**  
Read Personal  
History